


Sharing Your Reports

LibreView allows you to share glucose data with your healthcare providers

If you are uploading your reader data at home or using FreeStyle LibreLink app, your provider may ask you to share your glucose data with them.

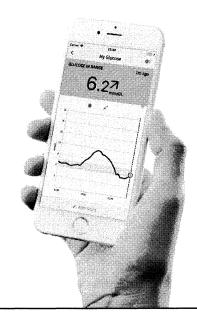





Download the FreeStyle LibreLink app on your iPhone* from Apple App Store

There are two ways to connect your account to your Health Care Provider. First, create your LibreView account at www.LibreView.com or through FreeStyle LibreLink app, then follow the directions below:

Option 1: Accept the invitation from your healthcare provider

- Your provider will ask for a current email address
- You will receive an email invitation to that email address
- Open the email and click "Accept"
- Sign in to your LibreView account and click "Accept"

Option 2: Log in manually with your healthcare provider's ID

- Your doctor will provide you with a "Practice ID" _ () すらいは子
- Log in to www.LibreView.com and navigate to "Account Settings" from the right navigation menu
- From the "Account Settings" page, click on the "My Practices" tab
- Input the "Practice ID" and click "Add"

Need Help? Contact Abbott Customer Care 1-888-205-8296 or visit the "Support" section on www.LibreView.com



LibreView is developed, distributed, and supported by NewYu, Inc. The LibreView data management software is intended for use by both patients and healthcare professionals to assist people with diabetes and their healthcare professionals in the review, analysis and evaluation of historical glucose meter data to support effective diabetes management. The LibreView software is not intended to provide treatment decisions or to be used as a substitute for professional healthcare advice. Trademarks are the property of their respective owners. Images are for illustration purposes only. Not actual patient data.

^{*}The FreeStyle LibreLink app is compatible with iPhone 7 and later running iOS 11 and later.